



QUICK REFERENCE GUIDE

ambIT[®] FAMILY OF PUMPS

For service or to place an order:

1-800-448-3569

844-4-AVANOS

painmanagement@avanos.com

For more information on the full line of Avanos pain management products, visit **avanospainmanagement.com**

AVANOS

avanospainmanagement.com



ambIT® electronic infusion therapy systems are simple to use, safe and offer a wide range of flexibility and customization. To program the pump to your patient's needs, just follow the steps below.

CLEAR HISTORY (HX0):

IMPORTANT: History must be cleared prior to programming a pump for a new patient.

On the PAUSE [II] screen, hold BOLUS [⌂] until you hear a beep, release, and immediately hold BOLUS [⌂] until you hear a second beep.

PROGRAMMING A PUMP:

- On the PAUSE [II] screen, simultaneously hold BOLUS [⌂] and RUN/PAUSE [▶/II] until you hear a beep.
- Select type of infusion.
 - For PCA Pump, continue to next step.
 - For PreSet Pump, select a PreSet program or PRO5 (Set Own).
 - For PIB/PCA Pump or PIB/IL, select PCA, PIB, or P+P.
- Press the RUN/PAUSE [▶/II] to increase or BOLUS [⌂] to decrease values as you program.
- Hold RUN/PAUSE [▶/II] to confirm number.

Note: Simultaneously, hold RUN/PAUSE [▶/II] and BOLUS [⌂] to quickly confirm and move to next screen.

Programming Screen Order

	Internal Limit*	Basal Rate	Intermittent Bolus Dose	Intermittent Bolus Lock Out	PCA Bolus Volume	PCA Lock Out	Infusion Volume
PCA Pump		1			2	3	4
PreSet Pump set to PRO5		1			2	3	4
PIB/PCA Pump set to PCA		1			2	3	4
PIB/PCA Pump set to PIB			1	2			3
PIB/PCA Pump set to P+P			1	2	3	4	5
PIB/IL Pump set to P+P	1		2	3	4	5	6

*Note: The Internal Limit maximum medication volume that will be dispensed over the Intermittent Bolus Lock Out Time.

ADD OR REMOVE PATIENT LOCK OUT

- On the PAUSE [II] screen, simultaneously hold BOLUS [⌂] and RUN/PAUSE [▶/II] until you hear a beep, release, and simultaneously press BOLUS [⌂] and RUN/PAUSE [▶/II].
- Enter Patient Lock Out Code of your choice.

Note: An icon [🔒] will be present on the screen once a Patient Lock Out is active.

REVIEW HISTORY (HX):

On the PAUSE [II] screen, hold BOLUS [⌂] until a beep occurs.

REVIEW PROGRAMMING (RX):

On the PAUSE [II] screen, press BOLUS [⌂].

Note: The first time the RUN/PAUSE [▶/II] is pressed after programming or a Patient Lock Out is added, the programming (Rx) will be displayed for review.

CASSETTE PRIMING

- Connect the cassette to the Medibag/IV bag using the short tubing line.
- Gently squeeze the Medibag/IV Bag forcing the air bubbles from the bag and local into the tubing/cassette.
- Remove the orange protective cover from the bottom of the cassette.
- Once primed snap the cassette in place, preventing free flow.
- Either by hand or using the orange protective cover on the bottom of the cassette spin the silver dot one full rotation.



ALARMS

DISPLAY	SOUND	ALARM	TO CLEAR
[🔔] + OCL	Constant Beeping	Occlusion Alarm	Press the RUN/PAUSE button to silence the pump and again to restart the pump. If unable to clear, call the clinical support line.
[🔔] + MA	Constant Tone	Cassette not attached	Press the RUN/PAUSE button to silence the alarm. Then ensure the cassette is properly attached and resume the infusion.
[🔋]	5 short beeps every 4 minutes	Low Battery	Replace the batteries as soon as possible.
[🔔] + [🔋]	Constant Tone	Dead Battery	Press the RUN/PAUSE button to silence the alarm. Replace the batteries immediately, the current program will resume once it's restarted.
[🔔] + MA1	Constant Tone	Malfunction	Press the RUN/PAUSE button to silence the alarm. Double check the cassette.
[🔔] + MA	Constant Tone	Malfunction	Press the RUN/PAUSE button to silence the alarm. Close the tubing clamp and check for kinks in the tubing and that the clamps are open.
[🔔] + EEE	10 short beeps	End of pump life	Replace pump to continue or start therapy. **PreSet pump only**